

Vesco Q

ENTERAL FEEDING PUMP®

SMART



- + Interactive Touch Screen
- + Self-Guided Programming
- + Lock Screen Prevents Unwanted Interruptions

SIMPLE



- + Easy Single-Handed Set Loading
- + Audible Click Ensures Correct Placement
- + Repeat Last Setting Enables Quick Programming & Consistent Feeds

STRONG



- + Feeds Thicker Formulas ($\leq 2.5\text{Kcal}$) & Blended Diets*
- + Drop, Scratch & Water Resistant
- + 24-Hour Battery Life

*Do not use non-prescribed home-prepared, blenderized, or liquidized foods in the feed sets. Only use commercially available, prepared enteral feeding solutions and blenderized diets prescribed by the clinician.

Quick Start Guide

1



Press  button on top of pump

2



Insert cassette into green opening then click into place

3




Press "Start New Feed" and follow prompts for self-guided programming


Tips

Keypad:

Touch number on screen for keypad display

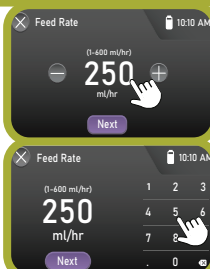

Nutrition Settings:

Go to Menu,  Liquid Type to change nutrition settings

