



















## Low Priority Alarms ● (Solid Yellow LED Indicator)

Alarm	Cause	Action
 Oops! Feed Set Not Attached	No Feed Set Attached.	Insert feed set cassette & click into place to begin feed.
 Quit Feed Setup	Feed Setup In Progress.	Quit or continue with a new feed program. Selecting <b>Exit (X)</b> at any point of displays this notification. Select <b>Quit Setup</b> to return to main menu or <b>Cancel</b> to continue programming.
 Maximum Limit Exceeded	Feed Rate Input Over Limit.	Value entered beyond maximum feed rate limit of 600 ml/hr. Enter value between 1 and 600 ml/hr.
 Maximum Limit Exceeded	Dose Input Over Limit.	Value entered beyond maximum total dose limit of 2000 ml. Enter value between 1 and 2000 ml.
 Minimum Limit Exceeded	Feed Rate Input Under Limit.	Value entered less than the minimum feed rate limit of 1 ml/hr. Enter value between 1 and 600 ml/hr.
 Minimum Limit Exceeded	Dose Input Under Limit.	Value entered less than the minimum total dose limit of 1 ml. Enter value between 1 and 2000 ml.
 Scheduled Feed	Feed Program Ready to Begin.	Reminder scheduled feed program is about to begin. Reminder appears 5 minutes before feed is scheduled to begin. Select <b>OK</b> to acknowledge alarm.
 System is Idle	Feed Program Paused for 20 Minutes with No Action.	Dismiss alert with <b>Cancel</b> or select <b>End Feed</b> to return to main menu. Note: After user selects <b>Cancel</b> , LED indicator will remain until feed resumes.
 Battery Low (20%)	Battery Low (20%).	Connect pump to charger & outlet soon. Monitor battery level with symbol in top right corner of screen.

## Medium Priority Alarms (Flashing Yellow LED Indicator)

Alarm	Cause	Action
 Occlusion Detected	Feed Line is Blocked.	Remove feed set cassette from pump & inspect set for any blockage or kinked tubing. Selecting <b>Continue</b> will silence the alarm. LED indicator will continue to flash until feed resumes.
 Air in the Line	Air Detected in Feed Set Line.	Check enteral fluid volume and tube connections. If alarm occurs more than 2 consecutive times, alarm will prompt mandatory prime of feed set.
 Alert	Feed Set Should Not be Connected to Patient While Priming.	Alarm appears before manual or automatic priming of pump set. Ensure feed set is not attached to patient before priming.
 Delete Feed	Delete Feed Confirmation.	Check before deleting a feed from schedule. Select <b>Delete</b> to confirm delete or select <b>Cancel</b> to cancel action.
 Feed Overlap	Current Feed Program Overlaps Scheduled Feed.	Select <b>Yes, Proceed</b> to continue with current feed, or select <b>Edit</b> to change feed program settings. To avoid overlapping feeds, check Scheduled Feeds before programming a new feed.
 Scheduled Feed	Feed Program is Ready to Begin.	Scheduled reminder appears when a scheduled feed program is ready to begin.
 End Feed?	Ending a Running Feed.	Selecting <b>Exit (X)</b> during the feed program displays the End Feed Prompt. Select <b>End Feed</b> to quit running feed or <b>Cancel</b> to continue feeding.
 Battery Low (5%)	Battery is Very Low (5%).	Connect pump to charger & outlet immediately. Monitor battery level with symbol in top right corner of screen. A depleted battery at 0% will take 6 hours to fully recharge.
 Alert	Critical System Error Occurred.	Record error code on bottom right of the screen for your service technician & contact Vesco Medical Technical Service. Note: When user selects <b>OK</b> , audible alarm will pause, but LED indicator will remain. If 30 seconds pass and alarm is not retrigged, LED will pause.

Please Contact The Vesco Q Technical Service Line for any questions or concerns.  
(888) 712-1905 or [techsupport@vescomedical.com](mailto:techsupport@vescomedical.com)