



# Mask/Device Compatibility List

This document specifies the masks compatible with ResMed's positive airway pressure devices and the mask setting, if one is required. For full details regarding the correct use of these devices, please refer to the specific User Guides.

Note: Not all products are available in all regions.

## Full Face Masks 1

Device	AcuCare™ F1-4	AirFit™ F40 <sup>4</sup>	AirTouch™ F30i <sup>1,4</sup>	AirFit™ X30i <sup>1,4</sup>	AirFit™ F30i <sup>1</sup> / AirFit™ F30i Non Magnetic <sup>1</sup>	AirFit™ F30	AirTouch™ F20 / AirTouch™ F20 for Her	AirFit™ F20 / AirFit™ F20 for Her / AirFit™ F20 Non Magnetic
AirCurve™ 11 VPAP Tx	X	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 VAuto / AirCurve™ 11 S	X	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 ASV	X	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirSense™ 11	N/V	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirMini™	X	X	X	X	X	✓	✓	✓
AirStart™ 10 / AirSense™ 10	N/V	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	N/V	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ASV	N/V	Pillows	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ST-A	N/V	Pillows <sup>3</sup>	Full Face <sup>3</sup>	Full Face <sup>3</sup>	Full Face <sup>3</sup>	Full Face <sup>3</sup>	Full Face	Full Face
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	N/V	Pillows	Full Face	Full Face	Full Face	N/V	Full Face	Full Face
S9 Escape™ / S9 Escape™ Auto	N/V	Pillows	Full Face	Full Face	Full Face	N/V	Full Face	Full Face
C-Series Tango™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
S7 Elite™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
S8 AutoSet Vantage™	N/V	N/V	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL
S8 Compact™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
S8 Elite™ / S8 Elite™ II	N/V	N/V	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL
S8 Escape™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
S8 Escape™ II	N/V	N/V	N/V	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.	
S8 AutoSet™ II	N/V	N/V	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL
VPAP™ III	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ III ST-A	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt SV	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	N/V	Pillows	Full Face	Full Face	Full Face	N/V	Full Face	Full Face

Device	AcuCare™ F1-4	AirFit™ F40 <sup>4</sup>	AirTouch™ F30i <sup>1,4</sup>	AirFit™ X30i <sup>1,4</sup>	AirFit™ F30i <sup>1</sup> / AirFit™ F30i Non Magnetic <sup>1</sup>	AirFit™ F30	AirTouch™ F20 / AirTouch™ F20 for Her	AirFit™ F20 / AirFit™ F20 for Her / AirFit™ F20 Non Magnetic
VPAP™ Adapt (S9 ASV)	N/V	Pillows	Full Face	Full Face	Full Face	N/V	Full Face	Full Face
VPAP™ Auto 25	N/V	N/V	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL
VPAP™ Auto (S9)	N/V	Pillows	Full Face	Full Face	Full Face	N/V	Full Face	Full Face
VPAP™ Malibu	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VPAP™ S / VPAP™ ST	N/V	N/V	N/V	N/V	N/V	N/V	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9)	N/V	Pillows	Full Face	Full Face	Full Face	N/V	Full Face	Full Face
VPAP™ ST-A (S9 iVAPS)	N/V	N/V	N/V	N/V	N/V	N/V	Full Face	Full Face
VPAP™ COPD (S9)	N/V	N/V	N/V	N/V	N/V	N/V	Full Face	Full Face
S9 VPAP™ Tx	N/V	Pillows <sup>2</sup>	Full Face <sup>2</sup>	Full Face <sup>2</sup>	Full Face <sup>2</sup>	N/V	Full Face	Full Face
VPAP™ Tx	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
Lumis™ Tx	N/V	N/V	N/V	N/V	N/V	N/V	Full Face	Full Face
Stellar™	N/V	N/V	N/V	N/V	N/V	N/V	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	✓	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V
Astral™	FULL FACE	N/V	N/V	N/V	N/V	N/V	FULL FACE	FULL FACE

<sup>1</sup> SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

<sup>2</sup> Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

<sup>3</sup> This combination of mask and device has been determined safe for use based on the evaluation of accuracy of monitoring of pressure, volume, minute ventilation and respiratory rate, and its alarms if applicable. Small variations in performance might be observed, particularly at higher pressure or flow. As per expected clinical practice, the effectiveness of ventilation and alarms should be verified by the clinician.

<sup>4</sup> SmartStart™ may not operate as described when using this mask with some CPAP or bi-level devices.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

## Full Face Masks 2

Device	AirFit™ F10 / AirFit™ F10 for Her	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Ultra Mirage™	Hospital Full Face
AirCurve™ 11 VPAP Tx	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 VAuto / AirCurve™ 11 S	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 11 ASV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirSense™ 11	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirMini™	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ASV	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
AirCurve™ 10 ST-A	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 Escape™ / S9 Escape™ Auto	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
C-Series Tango™			No setting required. Consult your clinician before changing to a different mask type.			
S7 Elite™	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL

Device	AirFit™ F10 / AirFit™ F10 for Her	Mirage Quattro™	Quattro™ FX / Quattro™ FX for Her Quattro™ Air / Quattro™ Air for Her	Mirage Liberty™	Ultra Mirage™	Hospital Full Face
S8 AutoSet Vantage™	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Compact™			No setting required. Consult your clinician before changing to a different mask type.			
S8 Elite™ / S8 Elite™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
S8 Escape™			Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.			
S8 Escape™ II			Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.			
S8 AutoSet™ II	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ III ST-A	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Adapt SV	FULL FACE	FULL FACE	FULL FACE	FULL FACE <sup>1</sup>	FULL FACE	FULL FACE
VPAP™ Adapt (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Adapt (S9 ASV)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Auto 25	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ Auto (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Malibu	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VPAP™ S / VPAP™ ST	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL	MIR FULL
VPAP™ S (S9) / VPAP™ ST (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ ST-A (S9 iVAPS)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ COPD (S9)	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
S9 VPAP™ Tx	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
VPAP™ Tx	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
Lumis™ Tx	Full Face	Full Face	Full Face	Full Face	Full Face	Full Face
Stellar™	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE
VS Ultra™ / VS Integra™	✓	✓	N/V	✓	✓	✓
VS III™	✓	✓	N/V	✓	✓	✓
Astral™	N/V	FULL FACE	FULL FACE	FULL FACE	FULL FACE	FULL FACE

## Nasal Masks 1

Device	AirTouch™ N30i <sup>1</sup>	AirFit™ N30 for AirMini	AirFit™ N30i <sup>1</sup> / AirFit™ N30	AirFit™ N20 / AirFit™ N20 for Her	AirTouch™ N20 / AirTouch™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage™ FX for Her	Hospital Nasal
AirCurve™ 11 VPAP Tx	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirCurve™ 11 VAUTO / AirCurve™ 11 S	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirCurve™ 11 ASV	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirSense™ 11	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirMini™	X	✓	X	✓	✓	X	X	X	X
AirStart™ 10 / AirSense™ 10	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirCurve™ 10 VAUTO / AirCurve™ 10 S / AirCurve™ 10 ST	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirCurve™ 10 ASV	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
AirCurve™ 10 ST-A	Pillows <sup>3</sup>	X	Pillows <sup>3</sup>	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal

Device	AirTouch™ N30i <sup>1</sup>	AirFit™ N30 for AirMini	AirFit™ N30i <sup>1</sup> / AirFit™ N30	AirFit™ N20 / AirFit™ N20 for Her	AirTouch™ N20 / AirTouch™ N20 for Her	AirFit™ N10 / AirFit™ N10 for Her	Swift™ FX Nano / Swift™ FX Nano for Her	Mirage™ FX / Mirage™ FX for Her	Hospital Nasal
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
S9 Escape™ / S9 Escape™ Auto	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
C-Series Tango™	N/V	X	N/V	N/V	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.	
S7 Elite™	N/V	X	N/V	N/V	N/V	N/V	N/V	STANDARD	STANDARD
S8 AutoSet Vantage™	N/V	X	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	STANDARD
S8 Compact™	N/V	X	N/V	N/V	N/V	No setting required. Consult your clinician before changing to a different mask type.			
S8 Elite™ / S8 Elite™ II	N/V	X	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	STANDARD
S8 Escape™	N/V	X	N/V	N/V	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.			
S8 Escape™ II	N/V	X	N/V	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.					
S8 AutoSet™ II	N/V	X	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	STANDARD
VPAP™ III	N/V	X	N/V	N/V	N/V	MIRAGE	MIRAGE	STANDARD	N/V
VPAP™ III ST-A	N/V	X	N/V	N/V	N/V	MIRAGE	MIRAGE	STANDARD	N/V
VPAP™ Adapt SV	N/V	X	N/V	N/V	N/V	N/V	N/V	VISTA <sup>1</sup>	VISTA
VPAP™ Adapt (S9)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
VPAP™ Adapt (S9 ASV)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
VPAP™ Auto 25	N/V	X	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	STANDARD
VPAP™ Auto (S9)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
VPAP™ Malibu	N/V	X	N/V	N/V	N/V	SWIFT	SWIFT	VISTA <sup>1</sup>	MERIDIAN
VPAP™ S / VPAP™ ST	N/V	X	N/V	SWIFT	SWIFT	SWIFT	SWIFT	STANDARD	STANDARD
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
VPAP™ ST-A (S9 iVAPS)	N/V	X	N/V	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
VPAP™ COPD (S9)	N/V	X	N/V	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
S9 VPAP™ Tx	Pillows <sup>2</sup>	X	Pillows <sup>2</sup>	Pillows	N/V	Pillows	Pillows	Nasal	Nasal
VPAP™ Tx	N/V	X	N/V	N/V	N/V	PILLOWS	PILLOWS	NASAL	NASAL
Lumis™ Tx	N/V	X	N/V	Pillows	Pillows	Pillows	Pillows	Nasal	Nasal
Stellar™	N/V	X	N/V	PILLOWS	PILLOWS	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>	NASAL	NASAL
VS Ultra™ / VS Integra™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	X	N/V	N/V	N/V	N/V	N/V	N/V	✓
Astral™	N/V	X	N/V	Pillows	Pillows	N/V	N/V	Nasal	Nasal

<sup>1</sup> SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

<sup>2</sup> Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

<sup>3</sup> This combination of mask and device has been determined safe for use based on the evaluation of accuracy of monitoring of pressure, volume, minute ventilation and respiratory rate, and its alarms if applicable. Small variations in performance might be observed, particularly at higher pressure or flow. As per expected clinical practice, the effectiveness of ventilation and alarms should be verified by the clinician.

N/V = The mask has not been validated for this device.

## Nasal Masks 2

Device	Mirage Micro™	Ultra Mirage™ II	Mirage Activa™	Mirage Activa™ LT	Mirage™ SoftGel	Mirage Vista™	Meridian	Mirage Kidsta™	Mirage Micro™ for Kids	Pixi™
AirCurve™ 11 VPAP Tx	X	X	X	Nasal	X	X	X	X	X	N/V
AirCurve™ 11 VAuto / AirCurve™ 11 S	X	X	X	Nasal	X	X	X	X	X	N/V
AirCurve™ 11 ASV	X	X	X	Nasal	X	X	X	X	X	N/V
AirSense™ 11	N/V	N/V	N/V	Nasal	N/V	N/V	N/V	N/V	N/V	N/V
AirMini™	X	X	X	X	X	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
AirCurve™ 10 ASV	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
AirCurve™ 10 ST-A	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
S9 Escape™ / S9 Escape™ Auto	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
C-Series Tango™	No setting required. Consult your clinician before changing to a different mask type.						N/V	N/V	N/V	N/V
S7 Elite™	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	STANDARD	N/V	N/V	N/V	N/V
S8 AutoSet Vantage™	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
S8 Compact™	No setting required. Consult your clinician before changing to a different mask type.						N/V	N/V	N/V	
S8 Elite™ / S8 Elite™ II	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
S8 Escape™	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.						N/V	N/V	N/V	
S8 Escape™ II	Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.						N/V	N/V	N/V	
S8 AutoSet™ II	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ III	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	STANDARD	N/V	N/V	N/V	N/V
VPAP™ III ST-A	STANDARD	ULTRA	STANDARD	STANDARD	STANDARD	STANDARD	N/V	STANDARD	STANDARD	N/V
VPAP™ Adapt SV	VISTA <sup>1</sup>	ULTRA	ACTIVA	ACTIVA	ACTIVA	VISTA	VISTA	N/V	N/V	N/V
VPAP™ Adapt (S9)	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ Adapt (S9 ASV)	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ Auto 25	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ Auto (S9)	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ Malibu	MERIDIAN	ULTRA	N/V	N/V	N/V	VISTA	MERIDIAN	N/V	N/V	N/V
VPAP™ S / VPAP™ ST	STANDARD	ULTRA	ACTIVA	ACTIVA	ACTIVA	STANDARD	STANDARD	N/V	N/V	N/V
VPAP™ S (S9) / VPAP™ ST (S9)	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	N/V
VPAP™ ST-A (S9 iVAPS)	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
VPAP™ COPD (S9)	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
S9 VPAP™ Tx	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
VPAP™ Tx	NASAL	NASAL-ULTRA	NASAL	NASAL	NASAL	NASAL	N/V	NASAL	N/V	N/V
Lumis™ Tx	Nasal	Nasal Ultra	Nasal	Nasal	Nasal	Nasal	Nasal	N/V	N/V	Pediatric
Stellar™	NASAL	ULTRA	NASAL	NASAL	NASAL	NASAL	N/V	NASAL	N/V	Pediatric
VS Ultra™ / VS Integra™	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V	N/V

Device	Mirage Micro™	Ultra Mirage™ II	Mirage Activa™	Mirage Activa™ LT	Mirage™ SoftGel	Mirage Vista™	Meridian	Mirage Kidsta™	Mirage Micro™ for Kids	Pixi™
VS III™	✓	✓	✓	✓	N/V	✓	N/V	✓	N/V	N/V
Astral™	Nasal	Nasal Ultra	N/V	Nasal	Nasal	Nasal	N/V	Nasal	Nasal	Pediatric

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible

## Nasal Pillows

Device	AirFit™ P30i <sup>1</sup>	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
AirCurve™ 11 VPAP Tx	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
AirCurve™ 11 VAuto / AirCurve™ 11 S	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
AirCurve™ 11 ASV	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
AirSense™ 11	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows
AirMini™	X	✓	X	X	X	X	X
AirStart™ 10 / AirSense™ 10	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 VAuto / AirCurve™ 10 S / AirCurve™ 10 ST	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 ASV	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
AirCurve™ 10 ST-A	Pillows <sup>4</sup>	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
S9 AutoSet™ / S9 AutoSet™ for Her / S9 Elite™	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows
S9 Escape™ / S9 Escape™ Auto	Pillows	X	Pillows	Pillows	Pillows	Pillows	Pillows
C-Series Tango™	N/V	X		No setting required. Consult your clinician before changing to a different mask type.			
S7 Elite™	N/V	X	MIRAGE	MIRAGE	MIRAGE	MIRAGE	MIRAGE
S8 AutoSet Vantage™	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Compact™	N/V	X		No setting required. Consult your clinician before changing to a different mask type.			
S8 Elite™ / S8 Elite™ II	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
S8 Escape™	N/V	X		Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.			
S8 Escape™ II	N/V	X		Requires pressure calibration at 12 cm H2O. Consult your clinician and check the user guide.			
S8 AutoSet™ II	N/V	X	SWIFT	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ III	N/V	X	MIRAGE <sup>2</sup>	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ III ST-A	N/V	X	MIRAGE <sup>2</sup>	MIRAGE	MIRAGE	MIRAGE	MIRAGE
VPAP™ Adapt SV	N/V	X	N/V	N/V	N/V	N/V	N/V
VPAP™ Adapt (S9)	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
VPAP™ Adapt (S9 ASV)	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
VPAP™ Auto 25	N/V	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ Auto (S9)	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
VPAP™ Malibu	N/V	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S / VPAP™ ST	N/V	X	SWIFT <sup>2</sup>	SWIFT	SWIFT	SWIFT	SWIFT
VPAP™ S (S9) / VPAP™ ST (S9)	Pillows	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
VPAP™ ST-A (S9 iVAPS)	N/V	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
VPAP™ COPD (S9)	N/V	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows

Device	AirFit™ P30i <sup>1</sup>	AirFit™ P10 For AirMini™	AirFit™ P10 / AirFit™ P10 for Her	Swift™ FX / Swift™ FX for Her	Swift™ FX Bella / Swift™ FX Bella Gray	Swift™ LT / Swift™ LT for Her	Mirage Swift™ II
S9 VPAP™ Tx	Pillows <sup>3</sup>	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
VPAP™ Tx	N/V	X	PILLOWS <sup>2</sup>	PILLOWS	PILLOWS	PILLOWS	PILLOWS
Lumis™ Tx	N/V	X	Pillows <sup>2</sup>	Pillows	Pillows	Pillows	Pillows
Stellar™	N/V	X	PILLOWS <sup>1,2</sup>	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>	PILLOWS <sup>1</sup>
VS Ultra™ / VS Integra™	N/V	X	N/V	N/V	N/V	N/V	N/V
VS III™	N/V	X	N/V	N/V	N/V	N/V	N/V
Astral™	N/V	X	N/V	Pillows	Pillows	Pillows	Pillows

<sup>1</sup> SmartStop may not operate as described when using this mask with some CPAP or bi-level devices.

<sup>2</sup> Accuracy of tidal volume/minute ventilation may reduce using the AirFit P10 mask in bilevel modes (S, T, ST), particularly at higher pressure supports.

<sup>3</sup> Excluding PAC and iVAPS modes. Only compatible for pressure range 4 - 25 cm H2O.

<sup>4</sup> This combination of mask and device has been determined safe for use based on the evaluation of accuracy of monitoring of pressure, volume, minute ventilation and respiratory rate, and its alarms if applicable. Small variations in performance might be observed, particularly at higher pressure or flow. As per expected clinical practice, the effectiveness of ventilation and alarms should be verified by the clinician.

N/V = The mask has not been validated for this device.

✓ = Compatible X = Not Compatible