

## InnoSpire Elegance USFAQ's

### Technology

Q. What type of compressor does InnoSpire Elegance use?

A. Piston.

Q. Is the InnoSpire Elegance suitable for use by children and adults?

A. Yes.

Q. Which drugs can be nebulized with InnoSpire Elegance?

A. Suitable for nebulizing most drugs that are licensed for aerosol therapy. Typically these are bronchodilators (albuterol), steroids (budesonide) or antibiotics (colistin). Sidestream Plus is usually recommended for antibiotics because a filter can be attached, (filter assembly can be purchased separately).

Q. Is InnoSpire Elegance approved for airline use?

A. No – InnoSpire Elegance is not a fully portable compressor and is therefore is not designed for use outside of the home or hospital, although could be transported in luggage. We produce a range of portable nebulizers – see our website for full details.

Q. What is the operational motor rating for InnoSpire?

A. InnoSpire Elegance is continuously rated. This means that it is suitable for use in either home or clinical environments and can be operated continuously without the need for a cooling period.

Q. What is the compressor life of InnoSpire?

A. Elegance is tested to 1000 hours of use with 5 year warranty.

### Treatment

Q. What is the treatment time?

A. Average treatment time for a 2.5ml dose: 6-8 mins\*. \*Using 2.5ml Salbutamol

### Technical specifications, Battery/Voltage

Q. What are the recommended operating, storage temperatures and atmospheric conditions?

A. Operating conditions

Temperature: 41° F to 104° F (5° C to 40° C)

Humidity: Min 10% RH - Max 95% RH Performance may vary depending on altitude above sea level, barometric pressure and temperature.

Q. What are the recommended Storage conditions?

A. Temperature: -13° F to 158° F (-25° C to 70° C)

Humidity: Min 10% RH - Max 95% RH.

Q. What are the AC power supply specifications? (What is the voltage of this product?)

A. AC Power: 120VAC / 60Hz

DC Power: not applicable.

Q. Is this unit battery operated and if so what type of battery is required?

A. It is not battery operated and can only be plugged in with an AC Adapter.

Q. Does the InnoSpire run from a battery?

A. No, InnoSpire Elegance is AC mains electricity operated only and cannot be operated through DC or battery.

## Cleaning and maintenance

Q. What maintenance is required with InnoSpire Elegance?

A. The exterior covers of the device may be cleaned as needed using a clean damp cloth. The storage area may also be wiped with a clean damp cloth. Please refer to the instructions for use for full details.

Q. What type of filter does InnoSpire Elegance use?

A. The filter is a bespoke design for InnoSpire by Filtrona.

Q. When should you change the filter?

A. Filter needs to be changed regularly (we recommend every 3 months or sooner if discolored or becomes wet) Handset should also be replaced.

## Service and support

Q. What is the warranty on InnoSpire Elegance?

A. 5 years. Please refer to the instructions for use for full details.

## Weights and Dimensions

Q. How much does the InnoSpire Elegance weigh?

A. 1.5kg

3.3 lb.

Q. What is the size of the InnoSpire Elegance? A.

165 x 165 x 108 mm

6.4" x 6.4" x 4.2".

Q. What is the noise level of InnoSpire?

A. 58 decibel.

## Accessories

Q. What accessories are supplied with InnoSpire Elegance?

A. **InnoSpire Elegance (US) Contents:** Reusable Sidestream, Disposable Sidestream, Air Supply Tubing, Mouthpiece, 1 filter and Multilanguage Instructions for Use including English, Spanish and French.

## Country of Origin

Q. Where is this product manufactured?

A. China.