

InnoSpire Mini US FAQ's

Technology

Q. Is it approved for use with children and adults?

A. Yes.

Treatment

Q. How long should my treatment take?

A. InnoSpire Mini 2.5ml treatment time with SideStream Plus is approx. 7 minutes*. *Using 2.5ml Salbutamol.

Q. Can I use my steroids and antibiotics with Mini as well as my salbutamol?

A. Yes, Mini is suitable for any drugs licensed for nebulization.

Battery/Voltage

Q. The battery won't charge up

A. Make sure the rubber terminal cover on the compressor has been moved aside to expose the connections.

Q. How long do I need to charge the battery for and how long will it last?

A. The battery will take 6 hours to charge, and should hold approx. 90 minutes of running time when fully charged.

Q. Can I use this device abroad?

A. No, Do not use any adapter, single or multiple, and/or extension cord.

Cleaning and maintenance

Q. Where is the filter on my device?

A. The filter is housed inside the filter cap (on the side of the device) which should be removed by turning it using a small coin. This releases the filter which can easily be changed. Please refer to the instructions for use for full details.

Service and support

Q. What is the warranty on Innospire Mini?

A. 3 years (compressor only, does not include battery).

Q. How long is the warranty on the battery?

A. The battery has 6 months, the InnoSpire Mini compressor has 3 years. The accessories do not have a warranty but have a 6 month life under normal use and if care instructions are followed. Please refer to the instructions for use for full details.

Weights and Dimensions

Q. How much does it weigh?

A. 608g (1.34 lb.) with battery, 420g (0.92lb) without battery.

Accessories

Q. What's included in the pack?

A. Sidestream Plus breath enhanced nebulizer, Mouthpiece, Air supply tubing, 4 spare filters, DC car adaptor, Rechargeable battery, Carry Case, Instructions for use.