

# PAP CLEANING GUIDE

RECOMMENDED CLEANING SOLUTION: WARM DRINKING QUALITY WATER AND UNSCENTED MILD SOAP

<u>SUPPLIES</u>	<u>HOW TO</u>	<u>FREQUENCY</u>
Mask Frame	Hand wash with cleaning solution Rinse thoroughly and allow to air dry	Daily
Headgear	Hand wash with cleaning solution Rinse thoroughly and allows to air dry	Weekly
Cushions Pillows	Wipe down with a damp cloth Daily Hand wash with cleaning solution weekly	Daily / Weekly
Tubing	Run cleaning solution through tubing Rinse thoroughly with warm water and hang to air dry	Weekly
Disposable Filters	Check Filters weekly, and brush off any build up Replace BI-Weekly	Weekly
Reusable Foam Filters	Check Filters Weekly, and rinse with cleaning solution, Allow filter to air dry Replace Every 6 months	Weekly
Water Chamber	Empty and replace distilled water daily Wash with cleaning solution weekly Rinse thoroughly and allow to air dry with lid open	Daily / Weekly

ONCE THE EQUIPMENT IS COMPLETELY DRY, REASSEMBLE IT AND RETURN TO USE

